



Vivid T8 R2.5



Product Description

The Vivid™ T8 is our Multipurpose Cardiovascular System designed for reliability in busy environments for cardiac and shared service imaging, with the capabilities of 2D, adult, pediatric, fetal/obstetrics, vascular/peripheral vascular, urological, abdominal, transcranial, small parts, musculoskeletal, and transesophageal applications.

System Architecture

Ease of Use features help make the Vivid T8 a productive cardiovascular ultrasound system – simplification of Vivid software and touch panel support working seamless in any environment. The Stress Echo, Auto EF and Scan Assist Pro options create a productive environment with added efficiency.

True Scan Raw Data is GE's innovative technology that allows for advanced processing on archived images by applying many of the same scan controls and advanced quantitative tools available during the original exam.

General Specifications

Dimensions and Weight

- Width:
 - Keyboard: 500 mm (19.7 in)
 - Caster: 720 mm (28.3 in)
- Depth:
 - Maximum: 810 mm (31.9 in)
 - Caster: 800 mm (31.5 in)
- Height:
 - Maximum: 1,565 mm (61.6 in)
 - Minimum: 1,415 mm (55.7 in)
- Weight: 60 kg, 132 lbs

Electrical Power

- Nominal input voltage: 100-240 VAC, frequency 50/60 Hz
- Power consumption maximum: 300 VA with peripherals

Console Design

- Four active probe ports
- ECG port
- Integrated HDD (500G)
- Multiple USB ports
- Optional DVD-RW drive
- Optional on-board storage for thermal printer, integrated speakers
- Integrated locking mechanism that provides rolling lock and caster swivel lock
- Integrated cable management
- Air filters easily accessible and removable for cleaning
- Front and rear handles
- Optional probe cable tray
- Integrated gel holder
- Six probe holders (four standard, two optional)

User Interface

Operator Keyboard

- Alphanumeric keyboard with support for eight character sets
- Ergonomic full size hard key layout
- Interactive back lighting for control panel
- Six TGC pods

Touch Screen

- 8.4" high-resolution, color, touch screen
- Interactive dynamic software menu

LCD Monitor

- 21.5" wide-screen, High-Definition (HD), flicker-free LCD display
- 256 shades of gray and 16.7 million simultaneous colors available
- Articulating monitor arm
 - LCD translation (independent of console)
 - 325 mm horizontal bidirectional
 - 150 mm vertical height adjustment
 - Swivel to side viewing direction
 - +25°/-90° vertical tilt on LCD
- Fold down and rotation lock mechanism for transportation
- Horizontal viewing angle of more than 170°
- Resolution: 1920 x 1080 pixels
- Digital brightness adjustment

System Overview

Operating System

- Windows® 7

Applications (probe dependent)

Abdominal, Cardiac (adult and pediatric), Musculo-skeletal (conventional and superficial), Small Organ, Pediatric, Obstetrics/Gyn, Fetal (heart and body), Transesophageal, Peripheral Vascular, Transvaginal, Transrectal, Adult, and Neonatal Cephalic

Scanning Methods

- Electronic sector
- Electronic convex
- Electronic linear
- CW pencil

Transducer Types

- Sector phased array
- Convex array
- Linear array
- CW Pencil
- Endovaginal
- TEE

Operating Modes

- 2D tissue
- 2D color flow
- 2D angio flow
- Color M-mode
- Tissue velocity M-mode
- Continuous wave Doppler
- Tissue M-mode
- Pulsed wave Doppler with high PRF
- Anatomical M-mode
- Curved anatomical M-mode (optional)
- Tissue velocity imaging
- Tissue tracking (optional)
- Tissue synchronization imaging (optional)
- Strain imaging (optional)
- Strain rate imaging (optional)
- Tissue Doppler imaging
- Blood flow imaging (optional)
- Blood flow angio flow imaging (optional)
- Smart Stress (optional)
- Auto EF (optional)

- AFI Automated Function Imaging (optional)
- Coded phase inversion LVO contrast (optional)
- Compound imaging
- Scan Assist Pro
- Scan Coach
- Image archive
- Z scores
- Fetal trending
- Renal calculations
- On-board report package

System Options

- Curved AMM
- Blood flow imaging
- Blood flow imaging with Angio
- B-flow
- Tissue tracking
- TSI
- Strain imaging
- Strain rate imaging
- Smart Stress
- AFI
- Auto EF
- Q-Analysis
- LOGIQ View
- IMT
- LVO contrast
- DICOM® connectivity
- DICOM viewer embedded on media
- Extra probe holder
- Paper tray
- Probe cable tray
- Flexible arm
- Smart Standby (optional)

Peripheral Options

- DVD RW drive
- Color video printer
- B/W video printer
- 4 GB memory stick
- 1 TB USB hard drive
- Three-pedal configurable footswitch

Display Modes

- Live and stored display format: Full size and split screen, both with thumbnails, for still and cine
- Review image format: 4 x 3 and thumbnails for still and cine
- Simultaneous capability
 - B + PW
 - B + CFM/TVI + PW
 - B + CFM/TVI
 - B + CFM/Angio/TVI/SRI/TT/SI/TSI (optional)
 - B + M/AMM/CAMM (optional)
 - B + CFM/Angio/TVI/SRI/TT/SI/TSI + M/AMM/CAMM (optional)
 - Real-time duplex or triplex mode
 - Compound + M/CFM/PW
 - B + color split screen (simultaneous mode)
- Selectable alternating modes
 - B or compound + PW
 - B + CW
 - B or compound + CFM/PW
 - B + CFM + CW
- Multi-image (split/quad screen)
 - Live and/or frozen
 - Independent cine playback
- Timeline display
 - Independent B (or compound) + PW/CW/M display
 - A choice of display formats with various sizes of 2D + PW/CW/M
- Top/bottom selectable format

Display Annotation

- Patient name: First, last and middle
- Patient ID
- Age, sex and birth date
- Hospital name
- Date format: Two types selectable
 - MM/DD/YY, DD/MM/YY
- Time format: Two types selectable
 - 24 hours, 12 hours
- Gestational age from LMP/EDD/GA
- Probe name
- Map names

- Probe orientation
- Depth scale marker
- Focal zone markers
- Image depth
- Zoom depth
- B-mode
 - Gain
 - Dynamic range
 - Imaging frequency
 - Frame averaging
 - Gray map
 - Speckle reduction imaging
 - UD clarity
- M-mode
 - Gain
 - Dynamic range
 - Time scale
- Doppler mode
 - Gain
 - Angle
 - Sample volume size and position
 - Wall filter
 - Velocity and/or frequency scale
 - Spectrum inversion
- Time scale
 - PRF
 - Doppler frequency
- Color flow Doppler mode
 - Frame rate
 - Frame averaging
 - Sample volume size
 - Color scale
 - Power
 - Color baseline
 - Color threshold marker
 - Color gain
 - PDI
- Acoustic frame rate
- CINE indicator, image number/frame number
- Bodymarks: Multiple human anatomical structures
- Application name
- Measurement results
- Operator message

- Displayed acoustic output
 - TIS: Thermal Index Soft Tissue
 - TIC: Thermal Index Cranial (Bone)
 - TIB: Thermal Index Bone
- MI: Mechanical Index
- Power output in dB
- Biopsy guide line and zone
- Heart rate
- Trackball-driven annotation arrows
- Active mode display
- Stress protocol parameters
- Parameter annotation follow ASE standard
- Free text with work library
- Image orientation marker

General System Parameters

System Setup

- Pre-programmable M&A and annotation categories
- Different user presets per probe/application may be stored for quick access
- User programmable preset capability with administrator preset protection
- Factory default preset data
- Languages: English, French, German, Spanish, Italian, Portuguese, Swedish, Danish, Dutch, Norwegian, Japanese, Chinese, Polish, Finnish, Greek, Russian, Hungarian, Slovak, Romanian, Czech, Latvian, Lithuanian, Turkish, Estonian, Korean, Serbian, Bulgarian, Croatian, Indonesian, Slovenia, Kazakhstan
- User-defined annotations
- Body patterns
- Customized comment home position

Comprehensive User Manual Available on Board

Available through touch panel utility page. User manual and service manual are included on CD with each system. A printed manual is available upon request.

CINE Memory/Image Memory

- 210 MB of cine memory
- Selectable cine sequence for cine review

- Measurements/calculations and annotations on cine playback
- Scrolling timeline memory
- Dual-image cine display
- Quad-image cine display
- CINE gauge and cine image number display
- CINE review loop
- CINE review speed

Image Storage

- On-board database of patient information from past exams
- Storage formats:
 - DICOM – compressed/uncompressed, single/multi-frame, with/without raw data
 - “Save As” JPEG, MPEG, AVI
- Storage devices:
 - USB memory stick: 4 GB
 - CD-RW storage: 700 MB
 - DVD storage: -R (4.7 GB)
 - Hard drive storage: 1 TB
- Compare old images with current exam
- Reload of archived data sets

Connectivity and DICOM

- Ethernet network connection
- DICOM (optional)
 - Verify
 - Print
 - Store
 - Modality worklist
 - Storage commitment
 - Modality Performed Procedure Step (MPPS)
 - Off-network DICOM spooler
 - Query/Retrieve
 - Structured reporting – compatible with adult cardiac and vascular
 - Media store of structured reporting
- DICOM media exchange
- InSite™ ExC capability for remote service/access

EchoPAC™ Connectivity

- Connectivity and image analysis capability of Vivid T8 from EchoPAC PC
- EchoPAC PC allows instant access to ultrasound raw data provided by the system
- Comprehensive review, analysis and post-processing capabilities on EchoPAC PC
- Advanced quantitative analysis and post-processing capabilities
- Q-analysis on raw data from Vivid T8 on EchoPAC PC
- Three user levels help organize data security requirements

Image and Data Management

- Exceptional workflow with instant access data management
- DICOM-SR standard structured reporting mechanism (optional)
- Next generation of DICOM Image Format: Raw image DICOM incorporates raw image data information with its data management flexibility into the image communication standard DICOM. Available on the scanner and EchoPAC.
- 2D, CFM or TVI data at maximum frame rate may be reviewed by scrolling or by running cine loops (can contain more than 1,000 images for imaging modes)
- Image clipboard for stamp-size storage and review of stored images and loops
- Built-in patient archive with images/loops, patient information, measurements and reports
- Structured findings report tools help support efficient text entries with direct editing of findings text, usability enhancements, new configuration options and conclusion section
- User can enter normal values which are then compared to actual measurements

- Configurable HTML-based report function
- Report templates can be customized on board
- ASE-based default text modules (English), user customizable
- Internal archive data can be exported to removable image storage through DICOM media
- Internal hard disk – for storing programs, application defaults, ultrasound images and patient archive
- All data storage is based on ultrasound raw data, allowing to change gain, baseline, color maps, sweep speeds, etc., for recalled images and loops
- DICOM media – read/write images on DICOM format
- Alphanumeric data can be exported in Microsoft® Excel® compatible format
- JPEG export for still frames
- AVI and MPEG export for cineloops

Self-contained DICOM Viewer (optional)

- Exams can be transferred to CD/DVD or USB media with an integrated “EZ DICOM CD viewer™”
- Self-contained “EZ DICOM CD viewer™” allows to review exams from media on a standard PC, without installing anything on the host PC

Scanning Parameters

- Digital beamformer with up to 193,536 effective digital channels
- Displayed imaging depth: 0 – 30 cm (probe dependent)
- Minimum depth of field: 0 – 2 cm (probe dependent)
- Maximum depth of field: 0 – 33 cm (probe dependent)
- Continuous dynamic receive focus/continuous dynamic receive aperture
- Dynamic range up to 252 dB, adjustable

- Image reverse: Right/left
- Image rotation of 0°, 180°

Tissue Imaging

General

- Variable transmit frequencies for resolution/penetration optimization
- Display zoom with zoom area control
- High-Resolution (HR) Zoom – concentrates all image acquisition power into selected Region of Interest (ROI)
- Variable contour filtering – for edge enhancement
- Depth range up to 30 cm – (probe specific)
- Selectable grayscale parameters (availability preset-dependent): Gain, reject, DDP, clarity, dynamic range and compress – can be adjusted in live, digital replay and image clipboard recall
- Automatically calculated TGC curves helps reduce operator interaction
- Automatically calculated lateral gain

2D Mode

- Sector tilt and width control
- Frame rate in excess of 1,000 fps, depending on probe, settings and applications
- Coded octave imaging with coded phase inversion – GE 3rd generation harmonic tissue imaging providing enhanced lateral and contrast resolution as compared to previous generation GE products. Features help reduce noise, help improve wall definition and axial resolution, making it well suited for a wide variety of patient groups
- Confocal imaging – allows for multiple transmit focal zones over range of view and a high vector density, probes dependent
- Automatic tissue optimization – single keystroke optimizes immediately and automatically different grayscale settings adjusted for the real time image

- UD Clarity and UD Speckle reduce imaging – an advanced image processing technique to help reduce speckle in real time examining the relative difference between neighboring pixel values and determining whether the grayscale variations have a sharp difference, follow a trend, or are random in nature
- Variable image width – a reduction either increases frame rate or increases the number of focal zones while maintaining the frame rate – application dependent
- Multiple-angle compound imaging – multiple co-planar images from different angles combined into a single image in real time to help improve border definition, contrast resolution, and reducing angular dependence of border or edge as compared to previous generation GE products
- LOGIQ View (optional) – provides the ability to construct and view a static 2D image with wider field-of-view of a given transducer. This allows viewing and measurements of anatomy that is larger than what would fit in a single image
- L/R and up/down invert, in live, digital replay or image clipboard recall
- Digital replay for retrospective review or automatic looping of images, allowing for adjustment of parameters such as gain, reject, anatomical M-mode, persistence and replay speed
- Data dependent processing performs temporal processing which helps reduce random noise but leaves motion of significant tissue structures largely unaffected – can be adjusted even in digital replay
- 256 shades of gray
- Colorized 2D-mode, user-selectable in real-time, digital replay

M-mode

- Trackball steers M-mode line available with all imaging probes – max steering angle is probe dependent

- Simultaneous real-time 2D- and M-mode
- M-mode – image data acquired is combined to give high-quality recording regardless of display scroll speed
- Digital replay for retrospective review of spectral data
- Several top-bottom formats, side-by-side format and time-motion only format – can be adjusted in live or digital replay
- Selectable horizontal scroll speed: 1, 2, 3, 4, 6, 8, 12, 16 seconds across display
- Horizontal scroll can be adjusted in live or digital replay

Anatomical M-mode

- M-mode cursor can be adjusted at any plane
- Curved anatomical M-mode – (optional) free (curved) drawing of M-mode generated from the cursor independent from the axial plane
- Can be activated from live, digital replay or image clipboard recall
- Anatomical color and tissue velocity M-mode
- M&A capability

Color Doppler Imaging

General

- Steerable color Doppler available with all imaging probes – max steering angle is probe dependent
- Trackball-controlled ROI
- Removal of color map from the tissue during digital replay
- Digital replay for retrospective review of color or color M-mode data allowing for adjustment of parameters such as encoding principle, color priority and color gain even on stored data
- PRF settings – user selectable
- Advanced regression wall filter gives efficient suppression of wall clutter

- For each encoding principle, multiple color maps can be selected in live and digital replay – variance maps available
- More than 65,000 simultaneous colors processed, providing a smooth display two-dimensional color maps containing a multitude of color hues
- Simultaneous display of grayscale 2D and 2D with color flow
- Color invert – user selectable in live and digital replay
- Variable color baseline – user selectable in live and digital replay
- Multi-variate color priority function gives delineation of disturbed flows even across bright areas of the 2D-mode image
- Color Doppler frequency can be changed independently from 2D

Color Flow Imaging

- TruSpeed imaging allows either ultra-high frame rate or increased lateral resolution as compared to previous generation GE products
- Frame Rate in excess of 150 fps, depending on probe and settings
- Variable ROI size in width and depth
- User-selectable radial and lateral averaging to help reduce statistical uncertainty in the color velocity and variance estimates
- Data Dependent Processing (DDP) performs temporal processing and display smoothing to help reduce loss of transient events of hemodynamic significance
- Digital replay for retrospective review or automatic looping of color images, allowing for adjustment of parameters such as DDP, encoding principle, baseline shift, color maps, color priority and color gain even on frozen/recalled data
- Application-dependent, multi-variate motion discriminator reduces flash artifacts

- Application-dependent, multi-variate motion discriminator helps reduce flash artifacts
- Dedicated coronary flow application

Color Angio

- Angle-independent mode for visualization of small vessels with enhanced sensitivity compared to standard color flow of previous GE products

Color M-mode

- Variable ROI length and position – user selectable
- User-selectable radial averaging to help reduce statistical uncertainty in the color velocity and variance estimates
- Selectable horizontal scroll speed: 1, 2, 3, 4, 6, 8, 12, 16 seconds across display – can be adjusted during live, digital replay or image clipboard recall
- Real-time 2D image while in color M-mode
- Same controls and functions available as in standard 2D color Doppler

Anatomical Color M-mode

- GE-patented, any plane color M-mode display derived from color Doppler cine loop
- Also applicable to tissue velocity imaging
- M&A capability

B-flow (optional)

- B-flow is a digital imaging technique that provides real-time visualization of vascular hemodynamics by directly visualizing blood reflectors and presenting this information in a grayscale display
- Use of GE-patented techniques to boost blood echoes, and to help preferentially suppress non-moving tissue signals
- B-flow is available for most vascular and shared service applications

Blood Flow Imaging (optional)

- Combines color Doppler with grayscale speckle imaging
- Helps improve delineation of blood flow without bleeding into tissue or vessel wall

Blood Flow Angio Imaging (optional)

- Combines angio with grayscale speckle imaging

Tissue Velocity Imaging

Tissue Velocity Imaging Mode

- Myocardial Doppler imaging with color overlay on tissue image
- Tissue Doppler data can be acquired in background during regular 2D imaging
- The velocity of myocardial segments after entire heart cycle can be displayed in one single image
- Tissue color overlay can be removed to show just the 2D image, still retaining the tissue velocity information
- Quantitative profiles for TVI, tissue tracking, strain and strain rate can be derived (optional)
- Time markers for valve events derived from any TM mode help simplify understanding of signals in velocity traces or curved anatomical M-mode

Tissue Tracking Mode (optional)

- Real-time display of the time integral of TVI for quantitative display of myocardial systolic displacement
- Myocardial displacement is calculated and displayed as a color-coded overlay on the grayscale and M-mode image – different colors represent different displacement ranges

Tissue Synchronization Imaging Mode (optional)

- Parametric imaging which gives information about synchronicity of myocardial motion
- Myocardial segments colored according to time to peak velocity, green for early and red for late peak

- Waveform trace available to obtain quantitative time to peak measurement from TSI Image
- Available in live scanning as well as an offline calculation derived from tissue Doppler data
- Additional features in combination with multi-dimensional imaging option
- Efficient segment specific TSI time measurements
- Immediate bull's eye report
- Automatic calculated TSI synchrony indexes
- TSI surface mapping
- LV synchronization report template
- CRT programming protocol

Strain/Strain Rate Mode (optional)

- Tissue deformation and rate of deformation are calculated and displayed as real-time, color-coded overlay on the 2D image
- Tissue deformation (strain) is calculated and displayed as real-time, color-coded overlay on the 2D Image
- Cine compound calculates and displays cineloops generated from a temporal averaging of multiple consecutive heart cycles
- Anatomical M-mode and curved anatomical M-mode (optional) displays (SI and SRI)

Spectral Doppler

General

- Operates in PW, HPRF and CW modes
- Trackball steerable Doppler available with all imaging probes – max steering angle is probe dependent
- Selectable Doppler frequency for enhanced optimization
- High-quality, real-time duplex or triplex operation in all Doppler modes, in PW, and for all velocity settings
- Frame rate control for optimized use of acquisition power between spectrum, 2D and color Doppler modes in duplex or triplex modes

- Very fast and flexible spectrum analysis with an equivalent DFT rate of 0.2 ms
- Dynamic gain compensation for display of flows with varying signal strengths over the cardiac cycle to help improve ease of use
- Dynamic reject gives consistent suppression of background – user selectable in real-time, digital replay or image clipboard recall
- Digital replay for retrospective review of spectral Doppler data
- Several top-bottom formats, side-by-side format and time motion only format – can be adjusted in live or digital replay
- Selectable horizontal scroll speed: 1, 2, 3, 4, 5, 6, 8, 12 seconds across display – can be adjusted in live or digital replay
- Adjustable spectral Doppler display parameters: Gain, reject, compress, color maps – can be adjusted in live or digital replay
- User-adjustable baseline shift – in live, digital replay and image clipboard recall
- Adjustable velocity scale
- Angle correction with automatic adjustment of velocity scale – in live, digital replay and image clipboard recall
- Stereo speakers mounted in the front panel
- Display annotations of frequency, mode, scales, Nyquist limit, wall filter setting, angle correction, acoustic power indices
- Compound in duplex

PW/HPRF Doppler

- Automatic HPRF Doppler maintains its sensitivity even for shallow depths and with high PRF's
- Digital velocity tracking Doppler employs processing in range and time for high-quality spectral displays
- Adjustable sample volume size of 1-8 mm (probe dependent)
- Maximum sample volume depth 30 cm

CW Doppler

- Highly sensitive steerable CW available with all phased array probes

Tissue Velocity Doppler

Contrast Imaging (optional)

LVO Contrast¹ – Enables contrast applications intended for imaging of the left ventricle:

¹ *Schering developed harmonic imaging for supporting contrast agent imaging.*

LV contrast (3Sc-RS probe) enhances delineation of the LV border in combination with ultrasound contrast agents. The new implementation of GE's Coded Phase Inversion (CPI) provides high-resolution detection of contrast in the LV cavity and excellent suppression of myocardial tissue signals.

Physiological Traces

- Integrated ECG or external ECG lead input
- High-resolution display of the ECG trace
- User adjustable trace gain/position and trace-invert controls
- User pre-settable trace gain/position control
- Automatic QRS complex detection with user ability to modify QRS trigger positions

Automatic Optimization

- Optimize B-mode image to help improve contrast resolution, gain, TGC and grayscale
- Auto-spectral optimize – adjustments baseline, PRF (on live image) and angle correction

Measurement and Analysis (M&A)

- Cardiac calculation package including extensive measurements and display of multiple repeated measurements
- Vascular measurements package

- Measurements can be labeled seamlessly by using protocols or post assignments
- Measurements assignable to protocol capability
- Parameter annotation follow ASE standard
- Seamless data storage and report creation
- User-assignable parameters
- Comprehensive set of cardiac measurements and calculations to help assess dimensions, flow properties and other functional parameters of the heart
- Comprehensive set of shared service measurements and calculations covering vascular, abdominal, obstetrics and other application areas
- Configuration package to set up a customized set and sequence of measurements to use, defining user-defined measurements and changing settings for the factory defined measurements
- Vivid stress echo support (optional) allowing wall motion scoring and automatic stress level labeling of measurements
- Support for measuring on DICOM images
- Automatic Doppler trace functionality for use in non-cardiac applications in both live and replay
- Worksheet for review, edit and deletion of performed measurements
- Reporting support allowing a configurable set of measurements to be shown in the exam report
- DICOM SR export of measurement data (optional)

Intima Media Thickness (IMT) Measurements (optional)

- Automated measurement of IMT rather than measuring the IMT manually
- Results representative of a region rather than a single point of a vessel wall

- Help reduce examination time by enhancing the procedure in measuring the IMT

Z-Scores

- Limited implementation of z-scores for a set of predefined pediatric dimension measurements

OB/GYN Application Module

- OB package for fetal growth analysis containing more than 100 biometry tables
- Dedicated OB/GYN reports
- Fetal graphical growth charts
- Growth percentiles
- Multi-gestational calculations (up to four)
- Programmable OB tables
- Expanded worksheets
- User-selectable fetal growth parameters based on European, American or Asian methods charts
- GYN package for ovary and uterus measurements and reporting
- Please refer to the user manual for the full list of measurements and calculations for all applications

Quantitative Analysis Package (Q-Analysis) (optional)

- Traces for tissue velocity or derived parameters (strain rate, strain, displacement) inside defined regions of interest as function of time
- Contrast analysis with traces for grayscale intensity or angio power inside defined regions of interest as function of time, including post processing ECG triggering and curve fitting for wash in/wash out analysis
- Curved anatomical M-mode display allowing an M-mode along an arbitrary curve in a 2D image

Automated Function Imaging (AFI) (optional)

- Parametric imaging tool which gives quantitative data for global and segmental wall motion

- Allows comprehensive assessment at a glance by combining three longitudinal views into one comprehensive bull's eye view
- Integrated into M&A package with specialized report templates
- 2D strain based data moves into clinical practice
- Simplified workflow with fully automated ROI tracings (if configured), quick tips and combined display of traces from all segments

Automated Ejection-Fraction Calc (optional)

- Automated EF measurement tool based on 2D-speckle tracking algorithm
- Integrated into M&A package with worksheet summary

Annotations

Body Marks

- Body mark icons for location and position of probe
- Easy selection of body marks from touch panel

Text Annotations

- Easy selection of text annotations from touch panel

Scan Assist Pro

- Customizable automations that assist the user through each step of the scan
- Helps enhance consistency and reduce keystrokes
- Supports selection of all modes, all measurements and dual annotations
- Imaging attributes: Octave, Steer, Dual/Quad screen, Compound, LOGIQ View, Zoom, Depth, Scale and Baseline
- On-line or off-line protocol editor
- Image acquisition according to predefined protocol templates
- Various factory protocol templates
- User configurable protocol templates

Stress Echo (optional)

Supported Protocol Examinations

- 2D pharmacological stress echo
- 2D bicycle stress echo
- 2D continuous capture stress echo (treadmill stress echo)
- Q-Stress protocols (acquire tissue velocity data in background for quantitative analysis)
- Cardiac resynchronization therapy programming protocols

Protocol Examinations May Include

- Wall motion scoring: Analysis by wall motion in individual myocardial segments
- Show reference: Show a reference image from baseline or previous level during acquisition
- Smart stress: Automatically set up various scanning parameters (for instance geometry, frequency, gain, etc.) according to same projection on previous level
- Scan mode settings: Scan mode may be specified for individual views in the protocol
- Preview of store: Show running loops as preview before storing to the examination

Continuous Capture

- Continuously acquire large amounts of 2D image data, and selection of projection views for analysis afterwards
- The entire continuous capture recording may be kept in memory while it is possible to store new images outside the protocol template, or the entire recording can be stored to file
- Selection of projection views on EchoPAC when the entire recording is stored to file

Wall Motion Scoring

- As part of the measurement and analysis package one can access a wall motion assessment module, providing analysis/scoring of individual myocardial segments
- For use with all stress modalities

Scan Coach

- Showing basic scanning techniques with graphic of probe position, schematic of anatomy and example clinical image

Smart Standby (optional)

- In case of accidental shutdown or power failure, the system automatically saves data and the system turns into "Standby" mode
- When power is restored, the system automatically turns on instantly, maintaining the exact system state prior to shutdown

Safety Conformance

The Vivid T8 is:

- CE Marked to Council Directive 93/42/EEC on Medical Devices

Conforms to the following standards for safety:

- IEC 60601-1 Medical electrical equipment – Part 1: General requirements for safety
- IEC 60601-1-2 Medical electrical equipment – Part 1-2: General requirements for safety
- Collateral Standard: Electromagnetic compatibility – requirements and tests EMC Emissions Group 1 Class A device requirements as per CISPR 11
- IEC 60601-2-37 Medical electrical equipment – Part 2-37: Particular requirements for the safety of ultrasonic medical diagnostic and monitoring equipment
- ISO 10993-1 Biological evaluation of medical devices – Part 1: Evaluation and testing
- EN 62366 Medical devices – Application of usability engineering to medical devices

Inputs and Outputs

- VGA output (SXGA resolution)
- Composite color video output via an adapter
- S-Video output via an adapter
- Audio stereo output

- 100BASE-TX ethernet (RJ45)
- USB (3x in rear, 3 under keyboard)

Virus Protection

- To reduce virus vulnerability, Vivid T8 is configured with a minimal set of open ports and with all network services not actively used by the system closed down. This helps to reduce the risk of a virus attack on Vivid T8.
- GE is continuously judging the need for additional actions to reduce vulnerability of equipment; this includes vulnerability scanning of our products and evaluation of new security patches for the third-party technology used. Microsoft (and other) security patches that address serious issues with Vivid T8 will be made available to customers after GE verification of those patches.

Transducers

3Sc-RS Phased Array Sector Probe

- Applications: Cardiac, cardiac_2, abdominal, renal, cranial, exercise, pediatric, LVO contrast, FAST, FATE
- Biopsy guide: Multi-angle, reusable bracket

6S-RS Phased Array Probe

- Applications: Pediatric, Pediatric_2, coronary, neonatal head, fetal heart, abdominal, pediatric abdominal

12S-RS Phased Array Probe

- Applications: Pediatric, pediatric_2, coronary, neonatal head, neonatal abdominal, carotid, pediatric abdominal

L6-12-RS Linear Array Probe

- Applications: Carotid, thyroid, breast, LEA, LEV, UEA, UEV, musculoskeletal, musculoskeletal superficial, small parts, scrotal, pediatric abdomen, pediatric hip, neonatal abdomen, neonatal head
- Biopsy guide: Multi-angle, disposable with a reusable bracket

9L-RS Linear Array Probe

- Applications: Carotid, thyroid, breast, LEA, LEV, UEA, UEV, musculoskeletal, musculoskeletal superficial, small parts, scrotal, pediatric abdomen, pediatric hip, neonatal abdomen, neonatal head, abdomen
- Biopsy guide: Multi-angle, disposable with a reusable bracket

12L-RS Linear Array Probe

- Applications: Carotid, thyroid, breast, LEA, LEV, UEA, UEV, musculoskeletal, musculoskeletal superficial, small parts, scrotal, pediatric abdomen, pediatric hip, neonatal abdomen, neonatal head
- Biopsy guide: Out of plane, multi-angle, disposable with a reusable bracket
- Infinite biopsy: Single angle disposable with a reusable bracket

L8-18i-RS Linear Array Probe

- Applications: Musculoskeletal, small parts, peripheral vascular

4C-RS Curved Array Probe

- Applications: Abdomen, abdomen 2, follicle, GYN, OB1, OB23, pediatric abdominal, prostate, renal
- Biopsy guide: Multi-angle, disposable with a reusable bracket

8C-RS Micro Convex Probe

- Applications: Pediatric abdominal, pediatric hip, neonatal abdominal, neonatal head, vascular

E8C-RS Endo Micro Convex Probe

- Applications: Fetal echo, follicle, GYN, OB1, prostate
- Biopsy guide: Fixed-angle, disposable, or reusable bracket

6Tc-RS TEE Probe

- Applications: Cardiac, cardiac_2, coronary

9T-RS TEE Probe

- Application: Cardiac

P2D Pencil Probe

- Applications: Cardiac

PROBE	FREQUENCY RANGE	CATALOG #
3Sc-RS	1.3 – 4.0 MHz	H45041DL
6S-RS	2.0 – 8.0 MHz	H45021RP
12S-RS	4.5 – 12.0 MHz	H44901AB
L6-12-RS	4.0 – 13.0 MHz	H48062AC
9L-RS	3.0 – 10.0 MHz	H40442LL
12L-RS	4.0 – 13.0 MHz	H40402LY
L8-18i-RS	4.5 – 18.0 MHz	H40462LF
4C-RS	1.8 – 6.0 MHz	H4000SR
8C-RS	3.5 – 11.0 MHz	H40402LS
E8C-RS	3.5 – 11.0 MHz	H40402LN
6Tc-RS	3.0 – 8.0 MHz	H45551ZE
9T-RS	4.0 – 10.0 MHz	H45531YM
P2D	1.9 MHz	H45551CA

Product may not be available in all countries and regions.
Full product technical specification is available upon request.
Contact a GE Healthcare Representative for more information.
Please visit www.gehealthcare.com/promotional-locations.

Data subject to change.

© 2017 General Electric Company. DOC1976333

GE, the GE Monogram, imagination at work, Vivid, EchoPAC and InSite are registered trademarks of General Electric Company.

DICOM is the registered trademark of the National Electrical Manufacturers Association for its standards publications relating to digital communications of medical information.

Microsoft, Windows and Excel are trademarks of Microsoft Corporation.

All other third-party trademarks are the property of their respective owners.

Reproduction in any form is forbidden without prior written permission from GE. Nothing in this material should be used to diagnose or treat any disease or condition. Readers must consult a healthcare professional.

About GE Healthcare

GE Healthcare provides transformational medical technologies and services to meet the demand for increased access, enhanced quality and more affordable healthcare around the world. GE (NYSE: GE) works on things that matter – great people and technologies taking on tough challenges. From medical imaging, software & IT, patient monitoring and diagnostics to drug discovery, biopharmaceutical manufacturing technologies and performance improvement solutions, GE Healthcare helps medical professionals deliver great healthcare to their patients.

GE Healthcare
9900 Innovation Drive
Wauwatosa, WI 53226
U.S.A.

www.gehealthcare.com

